




BinaxNOW[®] Malaria

The BinaxNOW[®] Malaria test is a rapid and simple diagnostic test that can accurately differentiate a deadly *P. falciparum* infection from a pan-malarial infection caused by *P. vivax*, *P. ovale* or *P. malariae*.

- 3 simple steps
- 1 reagent
- Reliable Results in **15 minutes**
- Sensitivity: 99.7% (P.f.); 93.5% (P.v.)*
- Specificity: 94.2% (P.f.); 99.8% (P.v.)

**Only FDA cleared
rapid malaria test!**

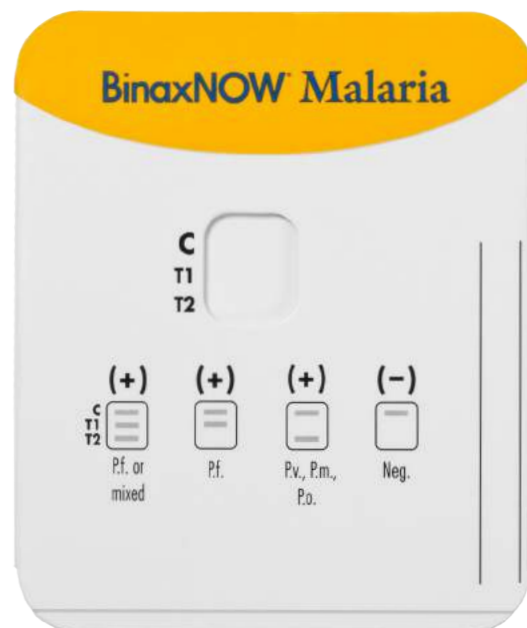


Actual size

To learn more about cost-effective, sensitive, rapid detection of **Malaria**, contact your local Alere representative.

Easy to Use

1

Apply 15 μ L of blood to the bottom half of the purple pad.

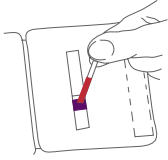


2

Apply 2 drops of Reagent to the white pad immediately below where the blood was applied.




3

Apply 4 drops of Reagent to the pad located at the top of the left hand side of the test card.

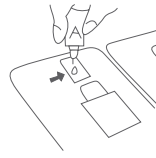

4

When the blood sample reaches the base of the white absorbent pad at the top of the test strip, close device and read at 15 minutes.

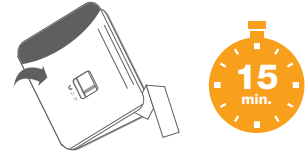

For complete instructions for use, see the package insert.

Easy to Read

Positive

P.f. or mixed

C
T1
T2



Positive

P.f.



Positive

P.v., P.m., P.o.



Negative



Order Information

BinaxNOW® Malaria Test

Moderately Complex
12 Test Kits

Order Number: 665-000

Moderately Complex
25 Test Kits

Order Number: 665-025

CPT® Code: 87899

BinaxNOW® Positive Control

Order Number: 665-010

Other BinaxNOW® Tests:

Alere BinaxNOW® Influenza A & B
Alere BinaxNOW® Legionella
Alere BinaxNOW® RSV
Alere BinaxNOW® *S. pneumoniae*
Alere BinaxNOW® *S. aureus*
BinaxNOW® Strep A
BinaxNOW® G6PD



Call **1.877.441.7440** or visit **alere.com** today

*For parasitemia levels >5,000 (parasites/ μ L). Please refer to the product insert for more information.

© 2013 Alere. All rights reserved. The Alere Logo, Alere and BinaxNOW are trademarks of the Alere group of companies. All other trademarks referenced are trademarks of their respective owners.
12000304-02 03/13